








CODE 34076

USER MANUAL - STRETCHER - FOLDABLE IN 2

SYMBOLS:

Symbol	Definition of Symbol
	ATTENTION: See Instructions for Use
	Symbol for "KEEP AWAY FROM SUNLIGHT"
	Symbol for "KEEP DRY"
	CE mark has been identified according to 93/42/EEC
	Symbol for "LOT NUMBER"
	Manufacturer
	Product code

I. Technical Features:

1. Product size (L x W x H): 188 x 50 x 3 cm
2. Folded size (L x W x H): 99 x 50 x 7 cm
3. Packing size (2pcs/carton): 96 x 54 x 16 cm
4. Net Weight: 4 kg Gross Weight: 9 kg
5. Load bearing: less than 159 kg

II. How to use:

This separable first - aid folding stretcher is mainly used for transporting patients with fractures. Both ends of the stretcher have hinge - type clutches to separate the stretcher into a right and left part. This allows for the patient to be quickly and gently placed onto the stretcher, operation table or bed. The stretcher is then withdrawn from beneath the patient.

III. Applied for:

Approved by clinic practice, this item is suitable for ambulance, hospital battlefield gym, to carry the wounded & patients. It also can be used in general operation.

IV. Cautions:

1. When carrying patients with this stretcher, be sure lock tightly the fender locking device & fasten the safety belt for safety sake.
2. in general operation, be careful not to out torn the stretcher surface.

V. Maintenance:

1. Keep clean as routine (incl. sterilization).
2. Often check if any parts loose or not.

VI. Storage & transportation:

1. Store this product in the place of damp proof & non-corrosion.
2. Common transporting vehicle can carry this product.

GIMA WARRANTY CONDITIONS

Congratulations for purchasing a GIMA product. This product meets high qualitative standards both as regards the material and the production. The warranty is valid for 12 months from the date of supply of GIMA. During the period of validity of the warranty, GIMA will repair and/or replace free of charge all the defected parts due to production reasons. Labor costs and personnel traveling expenses and packaging not included. All components subject to wear are not included in the warranty. The repair or replacement performed during the warranty period shall not extend the warranty. The warranty is void in the following cases: repairs performed by unauthorized personnel or with non-original spare parts, defects caused by negligence or incorrect use. The warranty is void if the above regulations are not observed and if the serial code (if available) has been removed, cancelled or changed. The defected products must be returned only to the dealer the product was purchased from. Products sent to GIMA will be rejected.



GIMA S.p.A. – Via Marconi, 1 – 20060 Gessate (MI)
Made in P.R.C.

Rev. 0.10.14