



PROFESSIONAL MEDICAL PRODUCTS

Gima S.p.A. – Via Marconi, 1 - 20060 Gessate (MI) Italy
Italia: tel 199 400 401 – fax 199 400 403 – gima@gimaitaly.com
Export: phone + 39 02 953854209/221/225 – fax + 39 02 95380056
- export@gimaitaly.com – www.gimaitaly.com

Code 27258

Electronic medium pet scale

Thank you for choosing our product.

Your new electronic petscale was designed for measuring your pet's weight for a many-year-long time period.

Please, read the instruction for use before using the scale.

Technical specification:

1" LCD display
Capacity: 100kg /220Lbs
Graduation: 100g / 0,2Lb
Auto off
Auto overload indicator ("ERR")
Low battery indicator ("LO")
kg, lb, St unit of measurement can be switched

User Guide

1. Before using the scale for the first time, please remove isolating tab from the battery at the backside of the scale.
2. Place the scale on a flat, hard level surface. On a soft surface (for example: on rug) the scale may show inaccurate value.
3. To start the scale, tap the scale platform with your foot.
4. For the first time the display shows a running "8" and after "0,0" appears on the display.
5. Wait until "0,0" is displayed!
6. Place pet on the scale.
7. Then your pet's weight will appear on the display flashing two times. The measured weight can be seen for about 10 seconds.
8. After weighing, it will automatically turn off approximately in 6 seconds.

Power:

The scale run on replaceable Lithium battery. (CR 2032) included

Replacing batteries:

When "LO" appears on the display, replace the batteries.

Error display:

Weighing above 100 kg, "ERR" is displayed.
Please do not overload the scale! In case of overloading the scale get damaged. In this case we have no liability for the product.

Environment:

The environmental temperature may influence the weighing accuracy. You can get correct weighing accuracy at room temperature (20-25°C)